



KneeRover[®] PRO – All Terrain User Manual CE

Congratulations on your purchase of the KneeRover[®] Pro All Terrain Scooter!

Experience the highest level of freedom and performance with the New KneeRover[®] Pro scooter - the first all terrain knee walker with an integrated shock for maximum comfort. Whether you are at home, at work, on the trail, or on the town, the KneeRover[®] Pro scooter will keep you on the move, safely navigating everyday life with ease and adding a little fun along the way.

Operating Instructions:

This medical device is designed to maximize mobility during your recovery from foot/ankle surgery or injury.

Place the knee of your injured leg on the knee platform and stand as straight as possible. Adjust the handlebar to waist height so the arms and hands are in a comfortable position. Adjust the knee platform so that your injured leg is supported at a 90 degree angle when standing.

Ensure that the hand brake is fully engaged before mounting the KneeRover[®] scooter. To move the scooter, simply push forwards or backwards with your good foot. Allow the scooter to glide, and as it slows down, push again.

Safety First:

The KneeRover[®] scooter is designed for easy steering and stopping and should be used to carefully move about at slow, controlled speeds. When turning the scooter you should have the brake applied and be moving at a slow, safe, controlled speed. It is easy to tip the scooter if you are making a turn with too much speed. Use caution and balance and operate at your own risk.

With routine use, nuts and bolts on the steering column, steering mechanism, and brake mechanism can and likely will become loose. Periodically check nuts and bolts after use and re-tighten as appropriate.

KneeRover® Set Up Instructions:

Please follow the step by step instructions on the following pages to properly assemble your KneeRover® All Terrain scooter. The diagram below identifies the key components of the scooter to aid with assembly. [Visit kneerover.com for assembly videos.](http://www.kneerover.com)


Weight Capacity = 350 pounds



Setup Instructions:


Step 1: Install Front Axle

Please take your time to carefully complete all steps before operating your new KneeRover® scooter. First, remove the kneepad from bottom of the scooter and put aside for step 4. Next, place the front axle with 12" wheels under the scooter frame, ensuring the round bolts by the wheels are facing up (as shown below). Align the axle and frame holes, insert front axle bolts and tighten completely using the allen wrench provided.

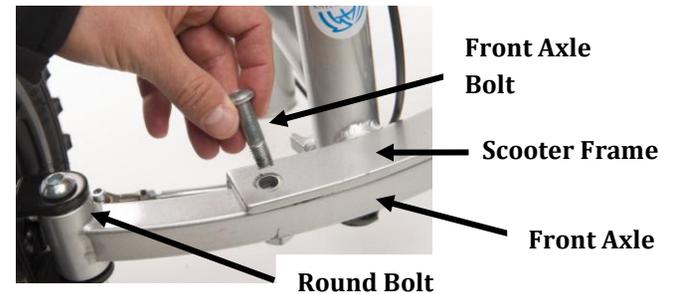

Step 2: Install Tie Rods

The fully assembled tie rods should be as follows (from top down): LEFT tie rod, washer, black guide, washer, RIGHT tie rod, washer, and nut. Insert the bolt into the top (LEFT) tie rod so that head of the bolt is sitting flush. Next insert the bolt thru the washer and black guide (as shown below). Place a washer and bottom (RIGHT) tie rod onto bolt from below the black guide. Finally insert a washer and nut, and then fully tighten.

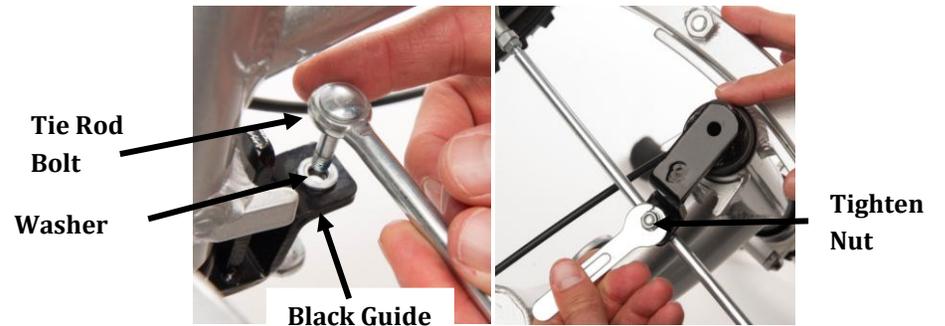

Step 3: Install Steering Column

The steering column will be in a folded position when unpacking. When facing the front of the KneeRover® scooter, raise the steering column and push the security pin in the clamp to the left to allow the column to lock into place. Raise the clamp lever upward and tighten by turning it clockwise. Finally, push clamp lever down firmly to secure the steering column. Reverse these steps when needing to lower steering column for transport.



To Lower Steering Column:
1) Lift Clamp Lever & Pull Toward You

2) Push Clamp Lever to Left to Release Security Pin

Step 4: Install Knee Platform & Inflate Tires

The knee platform is designed to be used with either the right or left leg. Insert the knee platform post into the receptacle tube and set the knee platform at your desired height for use by inserting the locking pin through the aligned holes. Finally, tighten the clamp to secure the knee platform in place.

The pneumatic tires come partially inflated. It is important to inflate all three tires to the correct pressure (a maximum of 40 psi) the same way you inflate a bike tire. Be careful not to over inflate the tires.



Step 5: Brake Operation & Adjustment

To engage the brake, squeeze the brake lever on handle bar. Your KneeRover® scooter comes equipped with a brake lock / parking brake. Simply squeeze brake handle firmly and press the button to engage the lock. To release the lock, squeeze the brake handle again.

If needed, the brake can be adjusted to avoid brake pad(s) from rubbing a spinning back wheel with a disengaged brake. Use the provided smaller allen wrench to make adjustments. Approaching the scooter from behind you can loosen or tighten the brake cable by loosening the top right bolt, adjusting brake cable, and re-tightening bolt.



Brake Lock



Top Right Bolt – Use Allen Wrench

Ongoing Maintenance

With routine use, nuts and bolts on the steering column, tie rod steering mechanism, and brake assembly can become loose. To ensure safe continuous operation of this KneeRover® scooter, it is important to periodically inspect the scooter for loose nuts and bolts and re-tighten as appropriate. Also, check the tire pressure routinely to ensure that tires are properly inflated up to 40 psi maximum. Be careful not to over inflate the tires. Please visit kneerover.com/pages/kneerover-maintenance-videos for additional support in maintaining your KneeRover..

Your Satisfaction is Our Top Priority:

Thank you for your business – we appreciate having you as a customer! We are committed to providing innovative and affordable mobility solutions to get you moving as soon as possible after injury or surgery and for the duration of your recovery – free to go about your busy, active life – and have a little fun doing it.

We hope that you are completely satisfied with your purchase. If you have any issues, please contact us and we will do everything possible to help you. Reviews have a significant impact on our small business. If you purchased this product on Amazon or eBay, we would greatly appreciate a review once you've had a chance to use your new KneeRover® scooter. We hope we've earned a 4 or 5 star review – if not, please give us the opportunity to help you before leaving a review. Thank you for your support. We appreciate you!

Need Help – Contact Us:

Phone: 877-737-9877

Email: help@kneerover.com

Web: www.kneerover.com

Assembly Support: kneerover.com/pages/assembly-video

EU AUTHORIZED REPRESENTATIVE

Murray's Medical Equipment Ltd
Airton Park, Airton Road, Tallaght,
Dublin 24 (Eircode D24 DY09), Ireland
T: 00 353 1 855 5696
www.murrays.ie



KneeRover® is a registered trademark of Rover Mobility, LLC.

Limited Lifetime Warranty

This warranty applies only to the original purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. Your KneeRover® brand product is warranted to be free of defects in materials and workmanship for a period of one year from the date of purchase. The side frames and cross braces are warranted for the lifetime of the KneeRover® scooter.

This warranty does not cover device failure due to owner misuse, negligence, or normal wear and tear. The warranty does not extend to non-durable components, which are subject to normal wear and need periodic replacement. These components include, but are not limited to tires, grips, brake pads and upholstery.

The warranty shall not apply for damages or defects caused by unauthorized parts, attachments or repairs, negligence, accident, or any circumstances beyond the control of KneeRover®, such as improper operation, maintenance or storage.

If you have any questions regarding your KneeRover® product or this warranty, please contact our Customer Care Team. We are available, for your convenience, toll free at 1-877-737-9877, Monday – Friday from 9 am to 5 pm Eastern.

Need Parts or Accessories?

Add an accessory like a knee pad cover, cup holder or headlight to enhance your experience on your new KneeRover® scooter. Keep your scooter operating in top shape with original manufacturer parts. Visit kneerover.com, to explore our full range of parts and accessories.