

Freedom Rollator

By Medline



MDS86825SL
MDS86825SLBC

MDS86825SLR
MDS86825SLSC

“Light as a loaf of bread and a gallon of milk!”

Weight Capacity: 250 lbs

SAVE THESE INSTRUCTIONS

Important information is contained in this booklet. Please read it carefully before using your Medline Rollator Rolling Walker. Consult your doctor, therapist or other qualified professional to determine if the Medline Rollator Rolling Walker will suit your needs.

Medline Industries, Inc. offers a Rollator that is manufactured from lightweight aluminum and is finished with a coat of rust-resistant, epoxy-based paint and has components and accessories that are manufactured from steel and durable plastic.



Seat Height adjustable from 18-22”
Handle Height adjustable 30-34”

SAFETY INSTRUCTIONS

TO ENSURE YOUR SAFETY IN USING THE MEDLINE ROLLATOR ROLLING WALKER, THE FOLLOWING SAFETY INFORMATION AND ALL INSTRUCTIONS MUST BE FOLLOWED.

GENERAL WARNINGS & SAFETY INSTRUCTIONS

WARNING!! Serious injury to the user and/or damage to the rollator walker frame or wheels may result from improper use.

- **DO NOT use this product without first reading and understanding the instructions contained in this booklet.** If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before use - otherwise serious bodily injury or product damage may occur.
- **DO NOT** use as a wheelchair.
- **DO NOT** attempt to push the rollator while someone is sitting in the seat.
- **DO NOT** self propel the rollator while seated. Serious injury to the user and/or damage to the rollator walker frame or wheels may result from improper use.
- **DO NOT** use the seat to transport people or objects. Do not use the seat to carry or move anything.
- **DO NOT** use the rollator if there is apparent damage or if any parts (other than the basket) are missing. If there are missing or damaged parts, please call Medline Customer Care toll free at 1-800-MEDLINE
- **DO NOT** exceed the maximum weight capacity (including basket contents) or serious injury could result: 250 lbs
- **DO NOT** sit on the seat when the rollator is on an incline. Serious risk of fall or injury may occur.
- **DO NOT** hang anything from the rollator handles or frame. This may cause the rollator to tip, resulting in bodily injury or rollator damage.
- **DO NOT** use the seat without the backrest in place. Falls are possible if the user leans or slides backward on the seat.
- **DO NOT** sit on the rollator seat unless the brakes are locked.
- **DO NOT** pull the backrest from side or back when the user is sitting on the rollator.
- **DO NOT** lift or carry the rollator by the backrest.
- **WHEN USING** the rollator in a stationary position, the hand brakes **MUST** be locked.
- **CONSULT WITH** your doctor, physical or occupational therapist or other qualified professional regarding the proper height adjustment for maximum support and correct brake activation.

- **BEFORE EACH** use make sure the rollator frame is fully opened and in a stable position. Check the brakes for proper operation. Make sure that all parts are secure and that moving parts are in good working order.
- **ALL WHEELS** must be in contact with the floor at **ALL TIMES**. This will ensure the rollator is properly balanced.
- **THE SEAT** must be in the down position when using the rollator for ambulatory assistance.

ASSEMBLY INSTRUCTIONS:

Note: You may wish to request a set up and operating demonstration with your local Medical Equipment provider or pharmacy.

Component list:



1 Rollator Frame + Rear Wheels (attached)



1 Backrest with Pushbuttons



2 Front Wheels



4 Triangular Hand Screws



1 Vinyl Bag

ASSEMBLY INSTRUCTIONS:

1. Carefully remove the Rollator components from the shipping carton.
2. Examine all parts for shipping damage. **DO NOT** use the rollator if there is apparent damage or if any parts (other than the basket) are missing. If there are missing or damaged parts, please call Medline Customer Care toll free at 1-800-MEDLINE.
3. Lay the Rollator down so that the seat is facing the floor. Straighten the legs until the Rollator unfolds.
4. Insert front wheel (L) into frame with the front sticker located above wheel facing forward. Slide caster in frame to the proper height and line up the holes. Secure in place using the triangular hand screw (I). **HAND TIGHTEN ONLY.** Repeat this step for the other front wheel.



5. Insert rear caster (B) into frame and slide up to the proper height. (should match the front) The brake cable should be lined up on the inside of the frame. Secure in place using the triangular hand screw (I). **HAND TIGHTEN ONLY.** Repeat this step for the other rear wheel.
6. Check all of the wheels to see that they are in good working order and roll freely.
7. Turn rollator over and place it upon its wheels. Check that the folding bracket (D) is fully extended. Place folding strap on top of vinyl bag. Push the seat down into place.
8. Insert the padded backrest (E) so that the padded portion faces the front of the unit. Push down on the buttons on the side of the backrest until the spring pins lock into place.
9. Place the basket under the seat. Secure snaps around front and back frame and also underneath basket to support rod (J).
10. Make sure that the folding strap under seat is placed on top of the basket.

A. Handgrips	G. Brake Cables
B. 8" Rear Wheels	H. Padded Seat
C. Handlebar	I. Triangular Hand Screw
D. Folding Bracket	J. Support Rod
E. Padded Backrest	K. Brake Lock
F. Push Button	L. Front Wheel

OPERATING INSTRUCTIONS

OPENING / FOLDING THE ROLLATOR

Opening

Unfold the Medline Rollator by pushing down on the support rod (J) to spread apart the frame until the folding brackets (D) are in full extension. Push the seat down into place.

Folding

Pull up on the seat. Locate folding strap and lift by strap.

CHECKING / ADJUSTING / OPERATING THE BRAKES

Before each use, check to make sure the brakes are working properly.



Pulling **up** on the brake handles locks the wheels momentarily.




Pushing **down** on the brake handles will lock the wheels in place.

To adjust the brakes:

1. Hold the brake pad against the wheel,
2. Pull up on the cable.
3. Adjust locking nut to proper setting.
4. Release brake pad.
5. Retest and repeat as necessary for good brake function.

2. Pull up on brake cable

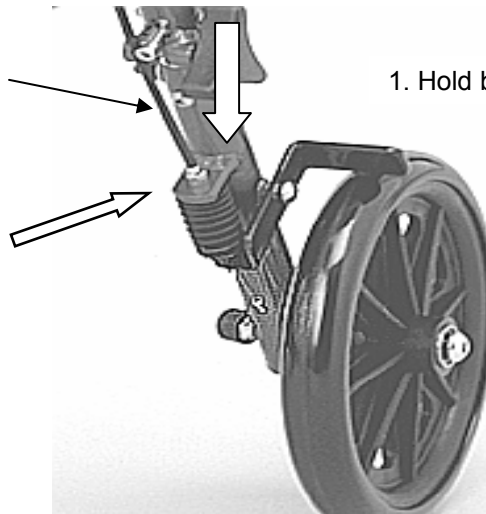

1. Hold brake pad against the wheel

3. Adjust locking nut to proper setting.

To Operate the Brakes:

- To slow down, squeeze the brake handles up.
- To lock the brakes, push down on the brake handles until you feel or hear the brakes click into locked position. If the brakes have been adjusted but the rear wheels on the rollator move while the brakes are in the locked position, **DO NOT** use the rolling walker. Please call our toll-free customer service number at 1-800-MEDLINE for returns or repairs.
- To release the brakes from the locked position, simply pull up on the brake handles.

To Use the Rollator as a Seat:

- Push the Medline Rollator into the position where you want to sit.
- **LOCK** the brakes **BEFORE** sitting.

MAINTENANCE:

Your Medline Rolling Walker should be checked periodically to ensure the brakes are functioning properly and that all nuts and bolts are secure.

Periodically test the brakes by both walking and squeezing the brakes and by stopping, pushing down on the brakes to a locked position, and then trying to push the walker. If the brakes do not lock the wheels, the brakes need to be adjusted. If adjustment does not tighten the brakes, **DO NOT** use the rolling walker.

Contact us at 1-800-MEDLINE, Monday – Friday 8am to 5pm CST if you encounter any problems or for information regarding repairs or maintenance.

Lifetime Limited Warranty

Your Medline brand product is warranted to be free of defects in material and workmanship for a lifetime for the original consumer. This device was built to precise standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the material and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse, negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, brakes and grips, which are subject to normal wear and replacement. If you have a question about your Medline product or this warranty, please call us at 1-800-MEDLINE. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

www.medline.com 1-800-MEDLINE

2007 All rights reserved. Medline is a registered trademark of Medline Industries, Inc.
Medline Industries, Inc., One Medline Place, Mundelein, IL 6060 USA