

OWNER'S MANUAL

Rollator

Stella



Before using this product, please
read this manual carefully before
to insure user safety

TABLE OF CONTENTS

General information.....	2
Specification and intended use.....	2
Warnings.....	3
Your rollator.....	4
Cleaning.....	4
Maintenance.....	4
Reuse by third parties.....	4
Using and adjusting your rollator.....	5-8
Technical specs.....	9
Limited warranty.....	9
Our mission.....	10
Our values.....	10
Contact us.....	10

GENERAL INFORMATION

These instructions for use are an integral part of the delivery of your rollator.

The user should keep them available at all times and keep them with the product if this is passed on to a third party.

The manufacturer reserves the right to make changes relative to the information provided in these instructions as a result of technical developments and improvements.

This document may not be reprinted, translated and/or duplicated in any form whatsoever, in part or in full, only subject to the manufacturer's prior written consent.

There is no update service for these instructions for use. Please contact the manufacturer to keep abreast of new developments.

SPECIFICATIONS AND INTENDED USE (FOR INDOOR AND OUTDOOR USE)

The **STELLA** rollator is intended as a walking aid for persons with reduced mobility who weigh up to 300 lb (136 kg) for the standard model and 400 lb (180 kg) for the heavy duty model. It also provides a limited seating function for temporary rest.

The rollator should be used only on firm, flat surfaces. It is not intended to be used for any other purpose such as a climbing aid or to carry heavy loads. Any such use can be dangerous!

WARNINGS : READ CAREFULLY BEFORE USING!

- ⚠ • Before sitting on the rollator, make sure the lumbar support is well positioned horizontally to prevent falling backwards. Be sure that both knobs are fully tightened at same height.
- ⚠ • Before sitting on the rollator, make sure the wheel locks are locked down.
- ⚠ • Before sitting on the rollator, put your hands on the hand grips and back-up until your legs touch the seat.
- ⚠ • Do not lean to one side, always keep your weight balanced on the rollator.
- ⚠ • Do not hang bags or other items from hand grips – risk of tipping over!
- ⚠ • Changing direction abruptly could make the rollator tip over.
- ⚠ • Use this rollator on flat surfaces only.
- ⚠ • Do not use the rollator to go up or down escalators.
- ⚠ • If excess weight is put onto the hand grips, the rollator could tip backwards.
- ⚠ • When folding or unfolding the rollator, watch for possible pinch points.
- ⚠ • Do not use your walking aid to replace a wheelchair or transport chair. When seated, the brakes should be applied.
- ⚠ • Test the brake handles to make sure the rollator brakes work properly.
- ⚠ • All the fabric exposed directly to the sun could become hot and burn your skin, make sure before using after a long exposition in the sun. We recommend you cover your rollator when in direct sun.

IF YOUR ROLLATOR BECOMES DEFECTIVE :

- ⚠ • Do not try to repair it yourself, contact your authorized dealer immediately.

YOUR ROLLATOR



1. Ergonomic handgrips
2. Brake handles
3. Back strap height adjustment
4. Handgrip height adjustment
5. Folding mechanism
6. Brake

CLEANING/MAINTENANCE/REUSE

Cleaning

Clean and disinfect the rollator at regular intervals. To do so, use mild soapy water with no scouring additives. You can disinfect the rollator using commercially available products. Never use steam degreasers. Never use scouring or harmful cleaning agents.

Maintenance

The **STELLA** rollator requires low-maintenance. If the braking function weakens, wheel movement becomes stiff or you notice any other functional impairment, please take the rollator to your authorized dealer for servicing.

Reuse by third parties

If you pass on your rollator to a new user, please remember also to hand over all the technical documentation needed to ensure safe operation. In accordance with specifications, the rollator must first be inspected, cleaned and returned to pristine condition by an authorized dealer.

USING AND ADJUSTING YOUR ROLLATOR

1. SETTING UP/UNFOLDING YOUR ROLLATOR



Place the rollator in front of you in the direction of travel and put the parking brake on.

Pull the seat strap to release the locking mechanism, press the tubes on either side of the seat down to the left and right until the folding locking mechanism clicks into place.

2. BRAKE HANDLES



Pull to brake (FIG. 1):


Pull up the brake handle to slow down or stop the rollator.



Push to lock (FIG. 2):

Push down the brake handle to lock the rollator in its position.

3. BRAKE ADJUSTMENT

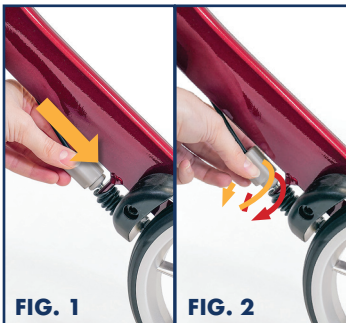

The brake is set correctly if there is no play in either the brake lever or the cable and if the wheels turn without rubbing.

The braking strength can be adjusted using the gray receiver by pushing it (FIG.1) and loosening or tightening the cable (FIG.2).


USING AND ADJUSTING YOUR ROLLATOR (CONTINUED)

4. CLIMBING SMALL OBSTACLES



Pressing your foot down on the extended frame makes it easier to lift the front wheels to negotiate curbs.


5. HANDLE HEIGHT ADJUSTMENT

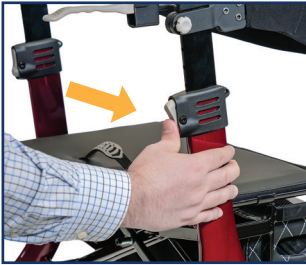

Activate the brake to immobilize the rollator.

Press the gray adjustment button until you obtain the required height which should be while standing, the wrist at the height of the handgrip. Once positioned, release the gray button.

6. BACK STRAP ADJUSTMENT

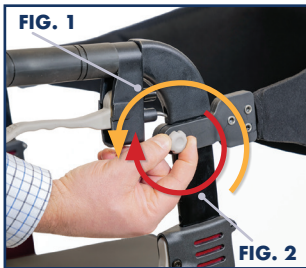

(FIG. 1):

Unscrew the gray knobs (on both sides of the rollator) and slide the back strap on the extruded handle until the required height. The back strap should be well positioned in the back in the lumbar area.

(FIG.2):

Once positioned, tighten the gray knobs.

USING AND ADJUSTING YOUR ROLLATOR (CONTINUED)

7. SITTING



Before sitting down, be sure that the rollator is on a level surface and lock the brake handles.

Before getting up, make sure the brake handles are still in lock position and place your hands on the handgrips as a support.

8. FOLDING YOUR ROLLATOR



Pull the strap in the middle of the seat and pull upwards (FIG.1) until the rollator locks in folded position (FIG.2).

9. ATTACHING/REMOVING THE CARRY BAG



To remove the carry bag, fold the rollator slightly (FIG.1) and slide bag out of receptacle pulling on both ends (FIG.2).

To replace the bag, reverse above steps.

The SWL (safe working load) is 4.5 kg (10 lb)



USING AND ADJUSTING YOUR ROLLATOR (CONTINUED)

10. CUP HOLDER INSTALLATION




(FIG.1) : The installation of the cup holder (*reversible left or right*) requires no tools. Simply clip it below the handle on the opposite side of the cane holder.

(FIG.2) : Adjust the cup holder direction according to your preference.

The cup holder (safe working load) is 1.3 kg (3 lb)

11. CANE HOLDER INSTALLATION

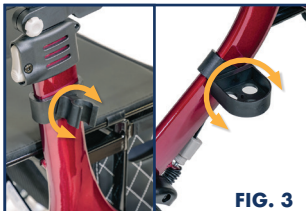

No tools are necessary for the installation of the cane holder (*reversible left or right*). Simply clip both cane holder's parts to the same side.

(Fig.1): Clip the cane holder's upper part underneath the handle.

(Fig.2): Clip the cane holder's bottom part above the brake or above the front wheel.

(FIG.3) : For proper cane alignment once clipped, both parts can be rotated to desired angle.

The cane holder SWL (*safe working load*) is 1.3 kg (3 lb)



TECHNICAL SPECS

SPECIFICATIONS	STELLA 18	STELLA 20	STELLA 23	STELLA HD
Seat height	18" (455 mm)	20" (510 mm)	23" (585 mm)	20" (510 mm)
Seat width	17.5" (445 mm)	17.5" (445 mm)	17.5" (445 mm)	19.5" (495 mm)
Overall width	24" (610 mm)	24" (610 mm)	24" (610 mm)	26" (660 mm)
Handle height	29.5" - 34.5" (750 mm - 875 mm)	30.5" - 35.5" (775 mm - 900 mm)	33.5" - 38.5" (850 mm - 980 mm)	30.5" - 35.5" (775 mm - 900 mm)
Overall length	28.5" (725 mm)	28.5" (725 mm)	28.5" (725 mm)	28.5" (725 mm)
Weight capacity	300 lb (136 kg)	300 lb (136 kg)	300 lb (136 kg)	400 lb (182 kg)
Weight	14.5 lb (6.6 kg)	14.5 lb (6.6 kg)	15 lb (6.8 kg)	16 lb (7.3 kg)

LIMITED WARRANTY

Amylior warrants this product to be free of defects in materials and workmanship.

The **STELLA** rollator has a lifetime warranty on the aluminum frame, 5 years on the brake handles and a 12-month warranty on all other parts. Parts subject to wear or misuse such as upholstery, handgrips and wheels are excluded from warranty.

If this product is found to be defective, it will be repaired or replaced according to the manufacturer's decision. This warranty does not cover freight or labor charges incurred during the repair or replacement of this product.

Amylior supplies products which respect the highest standards of quality. This product was assembled according to these standards and inspected carefully before delivery. This limited warranty represents our confidence in our materials and workmanship.

For service, contact the dealer where you purchased this product.

If for any reason, you are not satisfied, you may contact us directly. If you need to contact us, you must have available the product name, serial number, date of purchase and the dealer name.

DO NOT return any products without prior authorization. This warranty does not cover problems caused by improper or abusive use by the user.

OUR MISSION

Amylior's mission is to design and distribute rehabilitation products that will help improve the quality of life of people with a loss of autonomy.

To reach its goal, **Amylior** works closely with therapists, medical equipment specialists and end-users to meet the challenges people with a loss of autonomy face every day.

OUR VALUES

Amylior strives to improve the daily life of people with reduced mobility. That is the reason why **Amylior** offers affordable, high-quality products and unparalleled service.

Our corporate values speak for themselves:

- **Quality** : offering superior products and impeccable service at all times, without compromise.
- **Attentiveness** : meeting the real needs of people with reduced mobility.
- **Compassion** : understanding our customers with reduced mobility who use our products.

CONTACT US:

info@amylior.com

www.amylior.com



3190 F.-X.-Tessier
Vaudreuil-Dorion, QC
J7V 5V5, Canada
+1 450 424-0288