

# SUPINE TO PRONE IN ONE MOTION





EasyStandi

## Zing

MPS, Supine, MPS Tilt Table(TT),
Supine Tilt Table(TT), Vertical, & Prone

The Zing line of multi and single position standers were designed to fit the widest range of children and accommodate positioning challenges few others can. The Zing line features independent leg abduction up to 30 degrees on each model. The units can be configured simply, with padded strapping system to ease transfers and aid multi-user environments. But when required, individual positioning and support components can be added.



# The Zing

#### **True Multi-Position Stander**

- The MPS base allows three positions without changing pads, flipping foot plates or moving the tray
- Provides a smooth transition from a flat-to-load transfer to supine, vertical and 20 degrees prone
- Frame will only go supine and upright with Prone Position Lockout engaged

#### **True Hip Pivot Leg Abduction**

- Pivots at the hips for proper leg alignment in hips, knees and feet
- Feet maintain angle with the rest of the legs no wedges needed
- Up to 30 degrees of abduction per leg
- Promotes musculoskeletal development for children during critical growth years

#### **Plenty of Growth**

- Size 1 fits children from infancy to 44", Size 2 fits children 40"-60"
- Can easily grow with the child and adapt to their changing needs
- Size 1-70 pound weight capacity, Size 2-154 pound weight capacity

#### **Multi-Use Tray**

- The MPS, Supine, MPS TT and Supine TT feature a multi-use tray that is angle adjustable and swings away either direction
- Tray can remain attached during transfers
- More activities are possible by swinging tray away during standing therapy

# Advantage

#### **Accommodating Contractures**

- No special modifications are needed to accommodate children with contractures
- Using the multi-adjustable foot plates, multi-adjustable knee pads, and the independent calf pads, it is possible to adjust for contractures in one or both sides

#### **Hygienic Covers**

- Available for the pelvic, upper body, lateral, head and tray areas
- Washable
- · Especially beneficial if stander is used during feeding
- Keeps stander from getting soiled

#### **Unique Lift Mechanisms**

- Dual controls are available allowing angle adjustment by foot pedal or hand control
- Gas Spring Lockout prevents an inadvertent change of position, especially important when around other children
- The Pow'r Up Lift option on Size 2 brings the user to the standing position with the touch of a button

#### **Remarkably Modular**

- Can be configured as a simple stander or fit a child with the most complex needs
- Designed to grow with the child options can be added as needed
- Great for a multi-user facility or school





## Zing

MPS, Supine, MPS Tilt Table(TT),
Supine Tilt Table(TT), Vertical, & Prone










#### **Health Benefits**

- Improve/maintain range of motion
- Decrease joint/muscle contractures
- · Management of atrophy in the trunk and leg muscles
- Improve strength to trunk and lower extremities
- Decrease muscle spasms
- Promote proper muscle skeleton development
- Improve/maintain bone integrity/skeletal development
- Lessen/manage the progression of scoliosis
- Manage pressure (ulcers) through changing positions
- · Improve bowel function and regularity
- · Aid in kidney and bladder functions
- Strengthen cardiovascular system and build endurance
- Improve circulation
- Reduce swelling in lower extremities



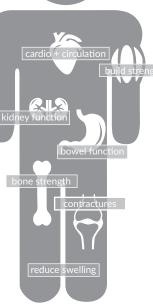

## **Hygienic Cover Patterns**

















### **Frame Colors**











## Zing-Size 1

MPS, Supine, MPS Tilt Table(TT), Supine Tilt Table(TT), Vertical, & Prone

additional specifications at easystand.com

SIZE

Weight Limit Height Range Transfer Height

> Supine Prone **HCPCS Positions** Tray Support **Dual Control**

Knee Pad Depth Range Weight of Basic Unit Frame Footprint

Gas Spring Lockout

Tool Free Adjustment

Transfer Aid

MPS (Multi-Position Stander)





The Zing MPS is the only multi-position stander that can seamlessly transition from supine to prone without having to transfer the child out and reposition foot plates, pads or trays.

The Zing Supine features a comfortable 32" transfer height and provides all the positioning and support features of the MPS.

The Zing MPS TT is identical to the MPS from a positioning standpoint, but features larger, wider support pads and options for padded strapping, that can simplify transferring and securing children.




atos, pado or trajo:		
70 lbs. (32kg)	70 lbs. (32kg)	70 lbs. (32kg)
up to 44" (112cm)	up to 44" (112cm)	up to 44" (112cm)
32"(81cm)	32" (81cm)	32" (81cm)
7"-22" (18-56cm)	7"-22" (18-56cm)	7"-22" (18-56cm)
4" (10cm)	4" (10cm)	4" (10cm)
5" (13cm)	5" (13cm)	5" (13cm)
55 lbs (25kg)	55 lbs (25kg)	55 lbs (25kg)
20"x28" (51x71cm)	20"x28" (51x71cm)	20"x28" (51x71cm)
Available	Available	Available
0-90°	0-90°	0-90°
0°-20°	n/a	0°-20°
E0641	E0638	E0641
Supine, Vertical, Prone	Supine, Vertical	Supine, Vertical, Prone
Adjustable, Swing-Away	Adjustable, Swing-Away	Adjustable, Swing-Away
dividual Positioning Components, Padded Strapping	Individual Positioning Components, Padded Strapping	Individual Positioning Components, Padded Strapping
Available	Available	Available
Available	Available	Available
Flat-to-Load	Flat-to-Load	Flat-to-Load
Yes	Yes	Yes





### FITTING THE ZING

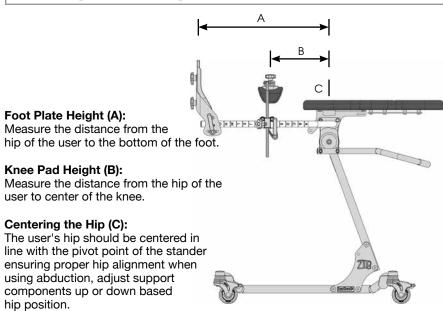

## Zing Supine/MPS (Multi-Position Stander) Size 1-Options

## Size 1

## 70 lbs (1,12cm) (32 kg)

#### The Supine/MPS (multi-position stander) base includes:

- Four Swivel Locking Casters
- Foot Operated Gas Spring
- Mast Receiver with Prone Position Lockout (MPS)



#### PA5522 Zing Supine Base PA5520 Zing MPS Base

Comes with four swivel locking casters, foot operated gas spring, mast receiver. MPS base comes with prone position lockout.



#### PA5624, PA5626, PA5628 Multi-Adjustable Foot Plates

Foot plates are adjustable forward/aft +/-.75" (2cm) +extra 1.5" (3.8cm), plantar/dorsi +/- 20°, toe-in/toe-out, swivel 30° left/right, and in height. Foot holder sizes: PA5624-6"Lx3"W (15x8cm), PA5626-7.75"Lx3.25"W (20x8cm), PA5628-9.75"Lx4"W (25x10cm).



#### PA5562 Planar Pads-5"Hx7"W PA5564 Planar Pads-5"Hx9"W

Cushioned pad covered with Neoprene®. Pad sizes: PA5562-5"Hx7"W (13x18cm), PA5564-5"Hx9"W (13x23cm).



#### PA5530 Mast

Mast is the base for positioning options including foot plates, knee supports, pelvic supports, upper body supports and head supports.



#### PA5540, PA5542, PA5544 Foot Holders for Platform Foot Plate

Use with the platform foot plate, gives added positioning of users feet. Foot holder sizes: PA5540-6"Lx3"W (15x8cm), PA5542-7.75"Lx3.25"W (20x8cm), PA5544-9.75"Lx4"W (25x10cm).



## PA5566 Form to Fit Pads-5"Hx10"W PA5570 Form to Fit Pads-5"Hx12"W

Form to fit pad can be custom fitted to the user. Pad sizes: PA5566-5"Hx10" (13x25cm) PA5570-5"Hx12"W (13x30cm). Not recommended with leg abduction mast.



#### PA5532 Mast with Leg Abduction

Mast is the base for support and positioning components and allows for 0-30° of independent leg abduction. Multi-adjustable knee pads and multi-adjustable foot plates required to do leg abduction.



#### PA5546 Secure Foot Straps-8"L PT50082 Secure Foot Straps-10"L

Velcro® and D-ring adjustment holds feet securely in place at ankles and toes with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



#### PA5574 Hip Supports

Keeps the hips in symmetrical alignment. Adjustable from 6"-10" (15-25cm) with fore/aft 4" (10cm). Pad size is 3.5"x3.5" (9x9cm). For planar pads only.



#### PA5612 Dual Control

Allows both foot and hand control of the gas spring lift. Must order push handle PA5594.



#### PA5548 Foot Straps-8"L PT50080 Foot Straps-10"L

Velcro® and D-ring adjustment holds feet in alignment with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



#### PA3011 Pelvic Positioning Belt

2" (5cm) wide belt has a D-Ring Velcro® closure, provides hip stability for user. Fits circumference up to 24" (61cm).



#### PA5614 Gas Spring Lockout

Locks gas spring in any position from lying down to fully upright, supine and prone positions.



#### PA5550 Calf/Knee Pad with Strap

4"x10" (10x25cm) pad attached to a 2" (5cm) belt with a D-Ring Velcro® closure, provides calf/knee stability for user. Fits circumference up to 24" (61cm). Gives support from the back of the calf/knee. Adjustable in height and fore/aft.



#### PA5610 Pelvic Padded Positioning Strap

5"x8" (13x20cm) pad attached to a 2" (5cm) belt. Fits circumference up to 24" (61cm).



#### PA5608 Directional Locking Caster

Great for transporting stander. Directionally locks one caster while leaving the other three free to swivel.



#### PA5556, PA5558, PA5729, PA5730 Multi-Adjustable Knee Pads

Multi-adjustable knee pads are adjustable in swivel, height +/-.75" (2cm), PA5556 inside width 2.5" (6cm), PA5558 inside width 3.25" (8cm), PA5729 inside width 4.25" (11cm) and PA5730 inside width 5" (13cm). With a depth range of 5" (13cm).



#### PA3028, PA3029 Hygienic Covers

Washable slip on cover available in six different patterns. For planar pads PA5562 and PA5564.



#### PA5536 Platform Foot Plate

Adjustable in height. Foot holders and foot straps can be added for more positioning of the feet.



#### PA5560 Multi-Adjustable Calf Pads 3.5" PA5678 Multi-Adjustable Calf Pads 4.5"

Great for adapting to knee contractures. Gives support from the back of the calf. Adjustable up and down with knee pad and fore/aft 4" (10cm) independently. Must order multi-adjustable knee pads.



#### PA3030, PA3032 Hygienic Covers

Washable slip on cover available in six different patterns. For form to fit pads PA5566 and PA5570.



PA5562 Planar Pads-5"Hx7"W PA5564 Planar Pads-5"Hx9"W

Cushioned pad covered with Neoprene®. Pad sizes: PA5562-5"Hx7"W (13x18cm), PA5564-5"Hx9"W (13x23cm).



PA5610 Upper Body Padded Positioning Strap 5"x8" (13x20cm) pad attached to a 2" (5cm) belt. Fits circumference up to 24" (61cm).



PA5600 Black Molded Swing-Away Tray-11"Lx19"W

Swing-away tray is accessible in all positions and provides anterior support as user moves from supine to upright and prone positions. Adjustable in depth, height and tray angle. Tray size is 11"Lx19"W (28x48cm).



PA5616 Planar Pads-7"Hx7"W PA5618 Planar Pads-7"Hx9"W

Cushioned pad covered with Neoprene®. Pad sizes: PA5616-7"Hx7"W (18x18cm), PA5618-7"Hx9"W (18x23cm).



PA3028, PA3029, PA3045, PA3046 Hygienic Covers

Washable slip on cover available in six different patterns. For planar pads PA5562, PA5564, PA5616, PA5618.



#### PA5602 Clear Swing Away Tray-11"Lx19"W

Swing-away tray is accessible in all positions and provides anterior support as user moves from supine to upright and prone positions. Adjustable in depth, height and tray angle. Tray size is 11"Lx19"W (28x48cm).



#### PA5568 Form to Fit Pads-5"Hx10"W PA5620 Form to Fit Pads-7"Hx10"W

Form to fit pad with built in lateral support that can be custom fitted to the user. Pad size for PA5568 is 5"Hx10"W (13x25cm) and PA5620 is 7"Hx10"W (18x25cm).



PA3031, PA3033, PA3047, PA3048 Hygienic Covers

Washable slip on cover available in six different patterns. For form to fit pads PA5568, PA5572, PA5620, PA5622.



#### PA5604 Forearm Wings-Black Molded Tray

Wings give additional area to rest arms when in standing position.



#### PA5572 Form to Fit Pads-5"Hx12"W PA5622 Form to Fit Pads-7"Hx12"W

Form to fit pad with built in lateral support that can be custom fitted to the user. Pad size for PA5572 is 5"Hx12"W (13x30cm) and PA5622 is 7"Hx12"W (18x30cm).



#### PA3039 Hygienic Covers for Lateral Supports

Washable slip on cover available in six different patterns. For lateral supports PA5576.



#### PA5606 Forearm Wings-Clear Acrylic Tray

Wings give additional area to rest arms when in standing position.



#### PA5576 Lateral Supports

Keeps the upper torso in symmetrical alignment. Adjustable from 7-11" (18-28cm) with a 3.5" (9cm) height range. Pad size is 3.5"Hx3.5"W (9x9cm). For planar pads only.



#### PY5626 Head Support-5"Hx8"W PY5628 Head Support-6"Hx10"W

Cushion size is 5"Hx8"W (13x20cm), 6"Hx10"W (15x25cm). Depth range is +1" to -2.75" (+2 to -7cm) from backrest. Maximum height from seat is 24" (61cm).



#### PA3038 Hygienic Padded Cover

Washable slip on cover is padded and available in six different patterns.



#### PA5578 Lateral Supports with Elbow Stop

Works as a lateral support and elbow rest. Great for the supine position. For planar pads only.



#### PA5586 Head Support-Form to Fit

Cushion size 5"Hx10"W (13x25cm). Depth range is -2.5" (6cm) to +1.5" (4cm) from backrest. Maximum height from top of upper body pad is 8" (20cm).



#### PNG50042 Hand Grips

Provides additional arm and hand support and alignment. Easily movable for precise arm positioning. Attaches with suction cups. Clear tray required.



#### PA5580

Lateral Supports w/Elbow Stop/Arm Rest

Works as a lateral support, elbow rest and armrest. Great for the supine, upright and prone position. For planar pads only.



#### PA3035, PA3036 Hygienic Covers

Washable slip on cover available in six different patterns. For PY5626 and PY5628.



#### PA5594 Push Handle

Makes pushing the stander easier for the caregiver and also aids in lifting and lowering the child. Required for dual control.



#### PA5630 Elbow Stop with Arm Rest

Great for loading and bringing to upright. Works with both planar and form to fit upper body supports. For planar and form to fit pads.



#### PA3037 Hygienic Cover

Washable slip on cover available in six different patterns. For PA5586.



#### P80809 Tool Pouch

Holds the assembly tools provided with the stander. Attaches to unit with Velcro® straps.



PT50086 X-Style Chest Vest-9.5"Lx9"W PT50088 X-Style Chest Vest-11"Lx9.5"W

Neoprene® with camlock adjustment provides anterior trunk support for child.



#### PA5596 Black Molded Tray-11"Lx19"W

Tray is accessible in all positions and provides anterior support as user moves from supine to upright and prone positions. Adjustable in depth and height. Tray size is 11"Lx19"W (28x48cm).



#### P80246 Angle Locator

Helps therapists monitor standing progress. Attaches to the mast. Can be used on leg abduction.

## Zing Supine/MPS Tilt Table (TT) Size 1-Options

# 70 lbs

#### The Supine/MPS (multi-position stander) TT base includes:

- Four Swivel Locking Casters
- Foot Operated Gas Spring
- Mast Receiver with Prone Position Lockout



#### PA5522 Zing Supine Base PA5520 Zing MPS Base

Comes with four swivel locking casters. foot operated gas spring, mast receiver. MPS base comes with prone position lockout.



#### PA5632 Mast

lea abduction.

PA5594.

positions.

PA5612 Dual Control

Mast is the base for support and positioning components.

PA5634 Mast with Leg Abduction

Mast is the base for support and

Multi-adjustable knee pads and

positioning components and allows for

multi-adjustable foot plates required to do

Allows both foot and hand control of the

Locks gas spring in any position from lying

down to fully upright, supine and prone

gas spring lift. Must order push handle

0-30° of independent leg abduction.



#### PA5624, PA5626, PA5628 Multi-Adjustable Foot Plates

Foot plates are adjustable forward/aft +/-.75" (2cm) +extra 1.5" (3.8cm), plantar/dorsi +/- 20°, toe-in/toe-out, swivel 30° left/right, and in height. Foot holder sizes: PA5624-6"Lx3"W (15x8cm), PA5626-7.75"Lx3.25"W (20x8cm), PA5628-9.75"Lx4"W (25x10cm).



#### PA5540, PA5542, PA5544 Foot Holders for Platform Foot Plate

Use with the platform foot plate, gives added positioning of users feet. Foot holder sizes: PA5540-6"Lx3"W (15x8cm), PA5542-7.75"Lx3.25"W (20x8cm). PA5544-9.75"Lx4"W (25x10cm).



### PA5546 Secure Foot Straps-8"L

securely in place at ankles and toes with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



#### Velcro® and D-ring adjustment holds feet



#### PA5548 Foot Straps-8"L PT50080 Foot Straps-10"L

Velcro® and D-ring adjustment holds feet in alignment with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



#### PA5644, PA5646, PA5648 Leg Support Planar Pads

Cushioned Neoprene® pad for leg support. Pad sizes:

PA5644-4"Hx6"W (10x15cm) PA5646-6"Hx6"W (15x15cm) PA5648-8"Hx6"W (20x15cm)



PA5644, PA5646, PA5648.



#### PA5666, PA5668, PA5670 Knee Strap

4"x6" (10x15cm) pad attached to a 1.5" (4cm) belt. Fits circumference up to 15" (38cm). Adjustable in height. For leg supports PA5644, PA5646, PA5648,



#### PA5556, PA5558, PA5729, PA5730 Multi-Adjustable Knee Pads

Multi-adjustable knee pads are adjustable in swivel, height +/-.75" (2cm), PA5556 inside width 2.5" (6cm), PA5558 inside width 3.25" (8cm), PA5729 inside width 4.25" (11cm) and PA5730 inside width 5" (13cm). With a depth range of 5" (13cm).



#### PA5560 Multi-Adjustable Calf Pads 3.5" PA5678 Multi-Adjustable Calf Pads 4.5"

Great for adapting to knee contractures. Gives support from the back of the calf. Adjustable up and down with knee pad and fore/aft -1"-4" (-2.5-10cm) independently. Must order multi-adjustable knee pads.



#### PA5638, PA5640, PA5642 Planar Pad

Cushioned pad covered with Neoprene®. Pad sizes:

PA5638-14"Hx14"W (36x36cm) PA5640-16"Hx14"W (41x36cm) PA5642-18"Hx14"W (46x36cm)



#### PA3063, PA3064, PA3065 Hygienic Covers

Washable slip on cover available in six different patterns. For planar pads PA5638, PA5640, PA5642.



#### PA5608 Directional Locking Caster

PA5614 Gas Spring Lockout

Great for transporting stander, Directionally locks one caster while leaving the other three free to swivel.



#### PA5660, PA5662, PA5664 Hygienic Covers

Washable slip on cover available in six different patterns. For planar pads



#### PA5636 Hip Supports

Keeps the hips in symmetrical alignment. Adjustable from 6"-10" (15-25cm) with fore/aft 4" (10cm). Pad size is 3.5"x3.5" (9x9cm).



#### PA5536 Platform Foot Plate

Adjustable in height. Foot holders and foot straps can be added for more positioning of the feet.



#### PA5550 Calf/Knee Pad with Strap

4"x10" (10x25cm) pad attached to a 2" (5cm) belt with a D-Ring Velcro® closure, provides calf/knee stability for user. Fits circumference up to 24" (61cm). Gives support from the back of the calf/knee. Adjustable in height and fore/aft.



#### PA3062 Pelvic Positioning Strap

2" (5cm) wide belt has a D-Ring Velcro® closure, provides hip stability for user. Fits circumference up to 24" (61cm).



PA5672 Pelvic Padded Positioning Strap

5"x8" (13x20cm) pad attached to a 2" (5cm) belt. Fits circumference up to 24" (61cm).



PA3066, PA3035, PA3036, PA3037 Hygienic Covers

Washable slip on cover available in six different patterns. For head supports PA5650, PY5626, PY5628, PA5586.



#### P80809 Tool Pouch

Holds the assembly tools provided with the stander. Attaches to unit with Velcro® straps.



PA5652 Lateral Supports-4"-8"Range PA5654 Lateral Supports-7"-11"Range Keeps the upper torso in symmetrical

Keeps the upper torso in symmetrical alignment. PA5652 Adjustable from 4"-8" (10-20cm), PA5654 Adjustable from 7"-11" (18-28cm) with a 3.5" (9cm) height range. Pad size is 3.5" Hx3.5" W (9x9cm).



#### PA5596 Black Molded Tray-11"Lx19"W

Tray is accessible in all positions and provides anterior support as user moves from supine to upright and prone positions. Adjustable in depth and height. Tray size is 11"Lx19"W (28x48cm).



#### P80246 Angle Locator

Helps therapists monitor standing progress. Attaches to the mast. Can be used on leg abduction.



#### PA5674

Upper Body Padded Positioning Strap 5"x8" (13x20cm) pad attached to a 2" (5cm) belt. Fits circumference up to 24" (61cm).



#### PA5600 Black Molded Swing-Away Tray-11"Lx19"W

Swing-away tray is accessible in all positions and provides anterior support as user moves from supine to upright and prone positions. Adjustable in depth, height and tray angle. Tray size is 11"Lx19"W (28x48cm).



#### PT50086 X-Style Chest Vest-9.5"Lx9"W PT50088 X-Style Chest Vest-11"Lx9.5"W

Neoprene® with camlock adjustment provides anterior trunk support for child.



#### PA5602 Clear Swing Away Tray-11"Lx19"W

Swing-away tray is accessible in all positions and provides anterior support as user moves from supine to upright and prone positions. Adjustable in depth, height and tray angle. Tray size is 11 "Lx19"W (28x48cm).



#### PA3039 Hygienic Covers for Lateral Supports

Washable slip on cover available in six different patterns. For lateral supports PA5652 and PA5654.



#### PA5604 Forearm Wings-Black Molded Tray

Wings give additional area to rest arms when in standing position.



#### PA5650 Planar Head Support

Cushioned Neoprene® pad for head support. Pad size 8"Hx8.5"W (21x22cm).



#### PA5606 Forearm Wings-Clear Acrylic Tray

Wings give additional area to rest arms when in standing position.



#### PY5626 Head Support-5"Hx8"W PY5628 Head Support-6"Hx10"W

Cushion size is 5"Hx8"W (13x20cm), 6"Hx10"W (15x25cm). Depth range is +1" to -2.75" (+2 to -7cm) from backrest. Maximum height from seat is 24" (61cm).



#### PA3038 Hygienic Padded Tray Cover

Washable slip on cover for tray is padded and available in six different patterns.



#### PA5586 Head Support-Form to Fit

Cushion size 5"Hx10"W (13x25cm). Depth range is -2.5" (6cm) to +1.5" (4cm) from backrest. Maximum height from top of upper body pad is 8" (20cm).



#### PNG50042 Hand Grips

Provides additional arm and hand support and alignment. Easily movable for precise arm positioning. Attaches with suction cups. Clear tray required.



#### PA5676 Headrest Mounting Bracket

For use when mounting an after market head support.



#### PA5594 Push Handle

Makes pushing the stander easier for the caregiver and also aids in lifting and lowering the child. Required for dual control.

## Zing Vertical Size 1-Options



#### The Vertical base includes:

- Four Swivel Locking Casters
- Mast Receiver



#### PA5524 Zing Vertical Base

Comes with four swivel locking casters and a mast receiver.



#### PA5546 Secure Foot Straps-8"L PT50082 Secure Foot Straps-10"L

Velcro® and D-ring adjustment holds feet securely in place at ankles and toes with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



#### PA5574 Hip Supports

Keeps the hips in symmetrical alignment. Adjustable from 6"-10" (15-25cm) with fore/aft 4" (10cm), Pad size is 3.5"x3.5" (9x9cm). For planar pads only.



#### PA5530 Mast

Mast is the base for support and positioning components.



#### PA5548 Foot Straps-8"L PT50080 Foot Straps-10"L

Velcro® and D-ring adjustment holds feet in alignment with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



#### PA3011 Pelvic Positioning Strap

2" (5cm) wide belt has a D-Ring Velcro® closure, provides hip stability for user. Fits circumference up to 24" (61cm).



#### PA5532 Mast with Leg Abduction

Mast is the base for support and positioning components and allows for 0-30° of independent leg abduction. Multi-adjustable knee pads and multi-adjustable foot plates required to do lea abduction.



#### PA5550 Calf/Knee Pad with Strap

4"x10" (10x25cm) pad attached to a 2" (5cm) belt with a D-Ring Velcro® closure, provides calf/knee stability for user. Fits circumference up to 24" (61cm). Gives support from the back of the calf/knee. Adjustable in height and fore/aft.



#### PA5610 Pelvic Padded Positioning Strap

5"x8" (13x20cm) pad attached to a 2" (5cm) belt. Fits circumference up to 24" (61cm).



#### PA5608 Directional Locking Caster

Great for transporting stander. Directionally locks one caster while leaving the other three free to swivel.



#### PA5688, PA5690, PA5727, PA5728 Multi-Adjustable Knee Pads

Multi-adjustable knee pads are adjustable in swivel, height +/-.75" (2cm), PA5688 inside width 2.5" (6cm), PA5690 inside width 3.25" (8cm), PA5727 inside width 4.25" (11cm) and PA5728 inside width 5" (13cm). With a depth range of 5" (13cm).



#### PA3092 Quick Strap

2" (5cm) wide belt has a airline style buckle for quick closure, provides hip stability for user. Fits circumference up to 24" (61cm).



#### PA5680 Platform Foot Plate

Adjustable in height. Foot holders and foot straps can be added for more positioning of the feet.



#### PA5726 Knee Strap

4"x6" (10x15cm) pad attached to a 1.5" (4cm) belt. Fits circumference up to 15" (38cm). Adjustable in height. For PA5688 and PA5690.



#### PA3093 Padded Quick Strap

5"x8" (13x20cm) pad attached to a 2" (5cm) belt has a airline style buckle for auick closure. Fits circumference up to 24" (61cm).



#### PA5624, PA5626, PA5628 Multi-Adjustable Foot Plates

Foot plates are adjustable forward/aft +/-.75" (2cm) +extra 1.5" (3.8cm), plantar/dorsi +/- 20°, toe-in/toe-out, swivel 30° left/right, and in height. Foot holder sizes: PA5624-6"Lx3"W (15x8cm), PA5626-7.75"Lx3.25"W (20x8cm), PA5628-9.75"Lx4"W (25x10cm).



#### PA5562, PA5564, PA5616, PA5618 Planar Pad

Cushioned pad covered with Neoprene® Pad sizes:

PA5562-5"Lx7"W (13x18cm) PA5564-5"Lx9"W (13x21cm) PA5616-7"Lx7"W (18x18cm) PA5618-7"Lx9"W (18x21cm)



#### PA3028, PA3029, PA3045, PA3046 Hygienic Cover

Washable slip on cover available in six different patterns. For planar pads PA5562, PA5564, PA5616, PA5618.



#### PA5540, PA5542, PA5544 Foot Holders for Platform Foot Plate

Use with the platform foot plate, gives added positioning of users feet. Foot holder sizes: PA5540-6"Lx3"W (15x8cm), PA5542-7,75"Lx3,25"W (20x8cm), PA5544-9.75"Lx4"W (25x10cm).



#### PA5566 Form to Fit Pads-5"Hx10"W PA5570 Form to Fit Pads-5"Hx12"W

Form to fit pad can be custom fitted to the user. Pad sizes: PA5566-5"Hx10" (13x25cm) PA5570-5"Hx12"W (13x30cm).



#### PA3030, PA3032 Hygienic Cover

Washable slip on cover available in six different patterns. For form to fit pads PA5566 and PA5570.



#### PA5692, PA5562, PA5564, PA5616, PA5618 Planar Pad

Cushioned pad covered with Neoprene®. Pad

PA5692-5"Hx5"W (13x13cm) PA5562-5"Hx7"W (13x18cm) PA5564-5"Hx9"W (13x21cm) PA5616-7"Hx7"W (18x18cm)







#### PA3085, PA3086, PA3087, PA3088 Hygienic Cover

Washable slip on cover available in six different patterns. For form to fit pads PA5694, PA5696, PA5698 and PA5700.



#### PNG50042 Hand Grips

Provides additional arm and hand support and alignment. Easily movable for precise arm positioning. Attaches with suction cups. Clear tray required.



Form to fit pad with built in lateral support that can be custom fitted to the user. Pad sizes are: PA5566-5"Hx10"W (13x25cm), PA5570-5"Hx12"W (13x30cm).



#### PA5706 Face Aperture-5"H PA5708 Face Aperture-7"H

Pad size is 9"Hx3"W (23x8cm). With a 5" (13cm) or 7" (18cm) range from the upper body support, individual face aperture pads are adjustable. Supports the head.



#### PA5720 Push Handle for Tray

Makes pushing the stander easier for the caregiver.



#### PA5694 Form to Fit Pads-5"Hx10"W PA5696 Form to Fit Pads-5"Hx12"W

Form to fit pad with extra sternum positioning. Built in lateral support that can be custom fitted to the user.. Pad sizes are: PA5694-5"Hx10"W (13x25cm), PA5696-5"Hx12"W (13x30cm).



#### PA5710 Hygienic Cover

Washable slip on cover available in six different patterns. For face apertures PA5706 and PA5708.



#### P80809 Tool Pouch

Holds the assembly tools provided with the stander. Attaches to unit with Velcro®



#### PA5698 Form to Fit Pads-7"Hx10"W PA5700 Form to Fit Pads-7"Hx12"W

Form to fit pad with extra sternum positioning. Built in lateral support that can be custom fitted to the user. Pad sizes are: PA5698-7"Hx10"W (18x25cm), PA5700-7"Hx12"W (18x30cm).



#### PA5712 Black Molded Tray-11"Lx19"W

Tray provides anterior support. Adjustable in depth and height. Tray size is 11"Lx19"W (28x48cm).



#### P80246 Angle Locator

Helps therapists monitor standing progress. Attaches to the mast. Can be used on leg abduction.



#### PA5702 Lateral Supports-4"-8"Range PA5704 Lateral Supports-7"-11"Range

Keeps the upper torso in symmetrical alignment. PA5702 Adjustable from 4"-8" (10-20cm), PA5704 Adjustable from 7"-11" (18-28cm) with a 3.5" (9cm) height range. Pad size is 3.5"Hx3.5"W (9x9cm). For planar pads only.



#### PA5716 Multi-Adjustable Black Molded Tray-11"Lx19"W

Tray provides anterior support. Adjustable in depth, height and tray angle. Tray size is 11"Lx19"W (28x48cm).



#### PA5610

Upper Body Padded Positioning Strap 5"x8" (13x20cm) pad attached to a 2" (5cm) belt. Fits circumference up to 24" (61cm).



#### PA5718 Multi-Adjustable Clear Tray-11"Lx19"W

Tray provides anterior support. Adjustable in depth, height and tray angle. Tray size is 11"Lx19"W (28x48cm).



#### PA3039 Hygienic Covers for Lateral Supports

Washable slip on cover available in six different patterns. For lateral supports PA5702 and PA5704.



#### PA5604 Forearm Wings-Black Molded Tray

Wings give additional area to rest arms when in standing position.



#### PA3084, PA3028, PA3029, PA3045, PA3046 Hygienic Cover

Washable slip on cover available in six different patterns. For planar pads PA5692, PA5562, PA5564, PA5616, and PA5618.



#### PA5606 Forearm Wings-Clear Acrylic Trav

Wings give additional area to rest arms when in standing position.



#### PA3030, PA3032 Hygienic Cover

Washable slip on cover available in six different patterns. For form to fit pads PA5566 and PA5570.



#### PA3038 Hygienic Padded Cover

Washable slip on cover for tray is padded and available in six different patterns.

## Size <sup>-</sup>



#### The Prone base includes:

- Four Swivel Locking Casters
- Foot Operated Gas Spring
- Mast Receiver



#### PA5522 Zing Prone Base

Comes with four swivel locking casters, foot operated gas spring, and a mast receiver.



#### PA5624, PA5626, PA5628 Multi-Adjustable Foot Plates

Foot plates are adjustable forward/aft +/-.75" (2cm) +extra 1.5" (3.8cm), plantar/dorsi +/- 20°, toe-in/toe-out, swivel 30° left/right, and in height. Foot holder sizes: PA5624-6"Lx3"W (15x8cm), PA5626-7.75"Lx3.25"W (20x8cm), PA5628-9.75"Lx4"W (25x10cm).



#### PA5562, PA5564, PA5616, PA5618 Planar Pad

Cushioned pad covered with Neoprene®. Pad sizes: PA5562-5"Lx7"W (13x18cm)

PA5562-5 LX7 W (13x18cm) PA5564-5"Lx9"W (13x21cm) PA5616-7"Lx7"W (18x18cm) PA5618-7"Lx9"W (18x21cm)

PA5616-7"Lx7"W (18x18: PA5618-7"Lx9"W (18x21: PA5566 Form to Fit Pads-



#### PA5530 Mast

Mast is the base for support and positioning components.



#### PA5540, PA5542, PA5544 Foot Holders for Platform Foot Plate

Use with the platform foot plate, gives added positioning of users feet. Foot holder sizes: PA5540-6"LX3"W (15x8cm), PA5542-7.75"LX3.25"W (20x8cm), PA5544-9.75"Lx4"W (25x10cm).



## PA5566 Form to Fit Pads-5"Hx10"W PA5570 Form to Fit Pads-5"Hx12"W

Form to fit pad can be custom fitted to the user. Pad sizes: PA5566-5"Hx10" (13x25cm )PA5570-5"Hx12"W (13x30cm).



#### PA5532 Mast with Leg Abduction

Mast is the base for support and positioning components and allows for 0-30° of independent leg abduction. Multi-adjustable knee pads and multi-adjustable foot plates required to do leg abduction.



#### PA5546 Secure Foot Straps-8"L PT50082 Secure Foot Straps-10"L

Velcro® and D-ring adjustment holds feet securely in place at ankles and toes with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



#### PA5574 Hip Supports

Keeps the hips in symmetrical alignment. Adjustable from 6"-10" (15-25cm) with fore/aft 4" (10cm). Pad size is 3.5"x3.5" (9x9cm). For planar pads only.



#### PA5725 Dual Control

Allows both foot and hand control of the gas spring lift. Must order push handle PA5720.



#### PA5548 Foot Straps-8"L PT50080 Foot Straps-10"L

Velcro® and D-ring adjustment holds feet in alignment with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



#### PA3011 Pelvic Positioning Strap

2" (5cm) wide belt has a D-Ring Velcro® closure, provides hip stability for user. Fits circumference up to 24" (61cm).



#### PA5614 Gas Spring Lift Lockout

Locks gas spring in any position from lying down to fully upright and supine positions.



#### PA5550 Calf/Knee Pad with Strap

4"x10" (10x25cm) pad attached to a 2" (5cm) belt with a D-Ring Velcro® closure, provides calf/knee stability for user. Fits circumference up to 24" (61cm). Gives support from the back of the calf/knee. Adjustable in height and fore/aft.



#### PA5610 Pelvic Padded Positioning Strap

5"x8" (13x20cm) pad attached to a 2" (5cm) belt. Fits circumference up to 24" (61cm).



#### PA5608 Directional Locking Caster

Great for transporting stander. Directionally locks one caster while leaving the other three free to swivel.



#### PA5688 PA5690, PA5727, PA5728 Multi-Adjustable Knee Pads

Multi-adjustable knee pads are adjustable in swivel, height +/-.75" (2cm), PA5688 inside width 2.5" (6cm), PA5690 inside width 3.25" (8cm), PA5727 inside width 4.25" (11cm) and PA5728 inside width 5" (13cm). With a depth range of 5" (13cm).



#### PA3092 Quick Strap

2" (5cm) wide belt has a airline style buckle for quick closure, provides hip stability for user. Fits circumference up to 24" (61cm).



#### PA5680 Platform Foot Plate

Adjustable in height. Foot holders and foot straps can be added for more positioning of the feet.



#### PA5726 Knee Strap

4"x6" (10x15cm) pad attached to a 1.5" (4cm) belt. Fits circumference up to 15" (38cm). Adjustable in height. For PA5688 and PA5690.



#### PA3093 Padded Quick Strap

5"x8" (13x20cm) pad attached to a 2" (5cm) belt has a airline style buckle for quick closure. Fits circumference up to 24" (61cm).



PA3028, PA3029, PA3045, PA3046 Hygienic Cover

Washable slip on cover available in six different patterns. For planar pads PA5562, PA5564, PA5616, PA5618.



PA3084, PA3028, PA3029, PA3045, PA3046 Hygienic Cover

Washable slip on cover available in six different patterns. For planar pads PA5692, PA5562, PA5564, PA5616, and PA5618.



#### PA5606 Forearm Wings-Clear Acrylic Tray

Wings give additional area to rest arms when in standing position.



#### PA3030, PA3032 Hygienic Cover

Washable slip on cover available in six different patterns. For form to fit pads PA5566 and PA5570.



#### PA3030, PA3032 Hygienic Cover

Washable slip on cover available in six different patterns. For form to fit pads PA5566 and PA5570.



#### PA3038 Hygienic Padded Cover

Washable slip on cover for tray is padded and available in six different patterns.



#### PA5692, PA5562, PA5564, PA5616, PA5618 Planar Pad

Cushioned pad covered with Neoprene®. Pad sizes:

SIZES: PA5692-5"Hx5"W (13x13cm) PA5562-5"Hx7"W (13x18cm) PA5564-5"Hx9"W (13x21cm) PA5616-7"Hx7"W (18x18cm) PA5618-7"Hx9"W (18x21cm)



#### PA3085, PA3086, PA3087, PA3088 Hygienic Cover

Washable slip on cover available in six different patterns. For form to fit pads PA5694, PA5696, PA5698 and PA5700.



#### PNG50042 Hand Grips

Provides additional arm and hand support and alignment. Easily movable for precise arm positioning. Attaches with suction cups.



## PA5566 Form to Fit Pads-5"Hx10"W PA5570 Form to Fit Pads-5"Hx12"W

Form to fit pad with extra sternum positioning. Built in lateral support that can be custom fitted to the user. Pad sizes are: PA5566 5"Hx10"W (13x25cm), PA5570 5"Hx12"W (13x30cm).



#### PA5706 Face Aperture-5"H PA5708 Face Aperture-7"H

Pad size is 9"Hx3"W (23x8cm). With a 5" (13cm) or 7" (18cm) range from the upper body support, individual face aperture pads are adjustable. Supports the head.



#### PA5720 Push Handle for Tray

Makes pushing the stander easier for the caregiver and also aids in lifting and lowering the child. Required for dual control.



## PA5694 Form to Fit Pads-5"Hx10"W PA5696 Form to Fit Pads-5"Hx12"W

Form to fit pad with extra sternum positioning. Built in lateral support that can be custom fitted to the user. Pad sizes are: PA5694-5"Hx10"W (13x25cm), PA5696-5"Hx12"W (13x30cm).



#### PA5710 Hygienic Cover

Washable slip on cover available in six different patterns. For face apertures PA5706 and PA5708.



#### P80809 Tool Pouch

Holds the assembly tools provided with the stander. Attaches to unit with Velcro® straps.



## PA5698 Form to Fit Pads-7"Hx10"W PA5700 Form to Fit Pads-7"Hx12"W

Form to fit pad with extra sternum positioning. Built in lateral support that can be custom fitted to the user. Pad sizes are: PA5698-7"Hx10"W (18x25cm), PA5700-7"Hx12"W (18x30cm).



#### PA5712 Black Molded Tray-11"Lx19"W

Tray provides anterior support. Adjustable in depth and height. Tray size is 11"Lx19"W (28x48cm).



#### P80246 Angle Locator

Helps therapists monitor standing progress. Attaches to the mast. Can be used on leg abduction.



#### PA5702 Lateral Supports-4"-8"Range PA5704 Lateral Supports-7"-11"Range

Keeps the upper torso in symmetrical alignment. PA5702 Adjustable from 4"-8" (10-20cm), PA5704 Adjustable from 7"-11" (18-28cm) with a 3.5" (9cm) height range. Pad size is 3.5"Hx3.5"W (9x9cm). For planar pads only.



#### PA5716 Multi-Adjustable Black Molded Tray-11"Lx19"W

Tray provides anterior support. Adjustable in depth, height and tray angle. Tray size is 11"Lx19"W (28x48cm).



#### PA5610

#### Upper Body Padded Positioning Strap

5"x8" (13x20cm) pad attached to a 2" (5cm) belt. Fits circumference up to 24" (61cm).



#### PA5718 Multi-Adjustable Clear Tray-11"Lx19"W

Tray provides anterior support. Adjustable in depth, height and tray angle. Tray size is 11"Lx19"W (28x48cm).



#### PA3039 Hygienic Covers for Lateral Supports

Washable slip on cover available in six different patterns. For lateral supports PA5702 and PA5704.



#### PA5604 Forearm Wings-Black Molded Tray

Wings give additional area to rest arms when in standing position.

## Zing-Size 2

MPS, Supine, MPS Tilt Table(TT), Supine Tilt Table(TT), Vertical, & Prone

additional specifications at easystand.com







The Zing Supine features a comfortable 31" transfer height and provides all the positioning and support features of the MPS.

The Zing MPS TT is identical to the MPS from a positioning standpoint, but features larger, wider support pads and options for padded strapping, that can simplify transferring and securing children.

	Weight Limit
	Height Range
	Transfer Height
	Hip to Foot Plate Range
	Upper Body Support Height Range
	Knee Pad Depth Range
	Weight of Basic Unit
	Frame Footprint
	30° Independent Leg Abduction
	Supine
	Prone
	HCPCS
GWO TO	Positions
	Tray
	Support
	Dual Control
	Gas Spring Lockout
	Transfer Aid
TO SHARE THE SHA	Tool Free Adjustment
	Pow'r Up Lift Option
Page 5 Zing MPS- Size 2	7

Weight Limit	154 lbs. (70kg)	154 lbs. (70kg)	154 lbs. (70kg)
Height Range	40"-60" (102-152cm)	40"-60" (102-152cm)	40"-60" (102-152cm)
Transfer Height	31" (79cm)	31" (79cm)	31" (79cm)
Hip to Foot Plate Range	34" (86cm)	34" (86cm)	34" (86cm)
Support Height Range	6" (15cm)	6" (15cm)	6" (15cm)
Knee Pad Depth Range	7" (18cm)	7" (18cm)	7" (18cm)
Weight of Basic Unit	82 lbs (37kg)	82 lbs (37kg)	82 lbs (37kg)
Frame Footprint	26.75"x39.75" (68x101cm)	26.75"x39.75" (68x101cm)	26.75"x39.75" (68x101cm)
pendent Leg Abduction	Available	Available	Available
Supine	0-90°	0-90°	0-90°
Prone	0°-20°	n/a	0°-20°
HCPCS	E0641	E0638	E0641
Positions	Supine, Vertical, Prone	Supine, Vertical	Supine, Vertical, Prone
Tray	Adjustable, Swing-Away	Adjustable, Swing-Away	Adjustable, Swing-Away
Support	Individual Positioning Components, Padded Strapping	Individual Positioning Components, Padded Strapping	Individual Positioning Components, Padded Strapping
Dual Control	Available	Available	Available
Gas Spring Lockout	Available	Available	Available
Transfer Aid	Flat-to-Load	Flat-to-Load	Flat-to-Load
Tool Free Adjustment	Yes	Yes	Yes
Pow'r Up Lift Option	Yes	Yes	Yes



## Supine Tilt Table (TT)

The Zing Supine TT features the larger pads and padded strapping options of the MPS TT and positioning from flat to load supine to vertical.

## **COMING** SOON

offer a flat-to-load transfer position Prone positioning from 0-90

154 lbs. (70kg)

40"-60" (102-152cm)

31" (79cm)

34" (86cm)

6" (15cm)

7" (18cm)

82 lbs (37kg)

26.75"x39.75" (68x101cm)

Available

0-90° n/a

E0638

Supine, Vertical

Adjustable, Swing-Away

Individual Positioning Components, Padded Strapping

Available

Available


Flat-to-Load

Yes

Yes



### FITTING THE ZING

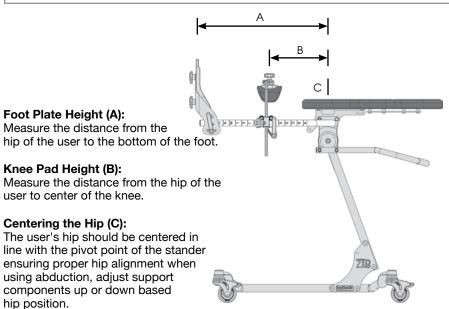

## Zing Supine/MPS (Multi-Position Stander) Size 2-Options

# Size 2 ",05" (154 lbs (70 kg)) (105-125cm)

#### The Supine/MPS (multi-position stander) base includes:

- Four Swivel Locking Casters
- Foot Operated Gas Spring or Pow'r Up Lift
- Mast Receiver with Prone Position Lockout (MPS Only)



## PB5502 Zing Supine Base Gas Spring Lift PB5500 Zing MPS Base Gas Spring Lift

Comes with four swivel locking casters, foot operated gas spring, mast receiver. MPS base comes with prone position lockout.



#### PB5518 Platform Foot Plate

Adjustable in height. Foot holders and foot straps can be added for more positioning of the feet. Platform 16.75"Wx11.5"L.



### PA5678 Multi-Adjustable Calf Pads 4.5" PB5536 Multi-Adjustable Calf Pads 5.5"

Great for adapting to knee contractures. Gives support from the back of the calf. Adjustable up and down with knee pad and fore/aft 4" (10cm) independently. Must order multi-adjustable knee pads.



## PB5506 Zing Supine Base Pow'r Up Lift PB5504 Zing MPS Base Pow'r Up Lift

Comes with four swivel locking casters, Pow'r Up Lift, mast receiver. MPS base comes with prone position lockout.



#### PB5520, PB5522, PB5524 Multi-Adjustable Foot Plates

Foot plates are adjustable forward/aft +/-.75" (2cm) +extra 1.5" (3.8cm), plantar/dorsi +/- 20°, toe-in/toe-out, swivel 30° left/right, and in height. Foot holder sizes: PB5520 7.75"Lx3.25"W (20x8cm), PB5522 9.75"Lx4"W (25x10cm). PB5524 11.75"Lx5"W (30x13cm).



#### PB3000 Planar Pad 10"Hx11"W PB3002 Planar Pad 10"Hx13"W

Cushioned pad covered with Neoprene®. Pad sizes:

PB3000-10"Hx11"W (25x28cm) PB3002-10"Hx13"W (25x33cm)



#### PB5512 Mast

Mast is the base for positioning options including foot plates, knee supports, pelvic supports, upper body supports and head supports.



#### PA5542, PA5544, PB5526 Foot Holders for Platform Foot Plate

Use with the platform foot plate, provides added positioning of users feet. Foot holder sizes: PA5542 7.75"Lx3.25"W (20x8cm),PA5544 9.75"Lx4"W (25x10cm), PB5526 11.75"Lx5"W (30x13cm).



## PB3004 Form to Fit Pad 10"Hx11"W PB3006 Form to Fit Pad 10"Hx13"W

Form to fit pad can be custom fitted to the user. Pad sizes: PB3004 10"Hx11" (25x28cm) PB3006 10"Hx13"W (25x33cm). Not recommended with leg abduction mast.



#### PB5514 Mast with Leg Abduction

Mast is the base for support and positioning components and allows for 0-30° of independent leg abduction. Multi-adjustable knee pads and multi-adjustable foot plates required with leg abduction.



#### PT50082 Secure Foot Straps-10"L PY5638 Secure Foot Straps-15"L PNG50506 Secure Foot Straps-21"L

Velcro® and D-ring adjustment holds feet securely in place at ankles and toes with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



#### PB5538, PB5540 Hip Supports

Keeps the hips in symmetrical alignment. Adjustable from PB5538 7"-13" (18-33cm) and PB5540 12"-18" (30-46cm) with fore/aft 4" (10cm). Pad size is 4.5"x4" (11x10cm). For planar pads only.



#### PB5516 Dual Control

Allows both foot and hand control of the gas spring lift. Must order push handle PB5560.



#### PT50080 Foot Straps-10"L PY5636 Foot Straps-15"L PNG50505 Foot Straps-21"L

Velcro® and D-ring adjustment holds feet in alignment with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



#### PA3011, PT30066 Pelvic Positioning Belt

2" (5cm) wide belt has a D-Ring Velcro® closure, provides hip stability for user. Fits circumference up to PA3011 29" (74cm) and PT30066 35" (89cm).



#### PA5614 Gas Spring Lift Lockout

Locks gas spring in any position from lying down to fully upright, supine and prone positions.



#### PB5528 Calf/Knee Pad with Strap

17.5"x5.5" (44x14cm) pad attached to a 2" (5cm) belt with a D-Ring Velcro® closure, provides calf/knee stability for user. Fits circumference up to 28"-38" (71-97cm). Gives support from the back of the calf/knee. Adjustable in height and fore/aft.



#### PA5610, PB5542 Pelvic Padded Positioning Strap

6"x10" (15x25cm) pad attached to a 2" (5cm) belt. Fits circumference up to PA5610 29" (74cm) and PB5542 35" (89cm).



#### PB5510 Directional Locking Caster

Great for transporting stander. Directionally locks one caster while leaving the other three free to swivel.



#### PB5530, PB5532, PB5534 Multi-Adjustable Knee Pads

Multi-adjustable knee pads are adjustable in swivel, height +/-.75" (2cm), PA5530 inside width 3.25" (8cm), PA5532 inside width 4.25" (11cm) and PB5534 inside width 5" (13cm). With a depth range of 7" (18cm).



#### PB3025, PB3026 Hygienic Cover

Washable slip on cover available in six different patterns. For planar pads PB3000 and PB3002.



PB3027, PB3028 Hygienic Cover

Washable slip on cover available in six different patterns. For form to fit pads PB3004 and PB3006.



PA5610 Upper Body Padded Positioning Strap PB5542 Upper Body Padded Positioning Strap

6"x10" (15x25cm) pad attached to a 2" (5cm) belt. Fits circumference up to PA5610 29" (74cm) and PB5542 35" (89cm).



#### PB5558 Clear Swing Away Tray-21"Lx24"W

Swing-away tray is accessible in all positions and provides anterior support as user moves from supine to upright and prone positions. Adjustable in depth, height and tray angle. Tray size is 21"Lx24"W (53x61cm).



PB3001 Planar Pad-8"Hx11"W PB3003 Planar Pad-8"Hx13"W

Cushioned pad covered with Neoprene®. Pad size for PB3001 is 8"Hx11"W (20x28cm) and PB3003 is 8"Hx13"W (20x33cm).



PB3029, PB3030 Hygienic Cover

Washable slip on cover available in six different patterns. For planar pads PB3301 and PB3003.



#### PT50156 Elbow Pad-9.25" Cutout

Additional padded support surface on the black molded or clear tray. 9.25" (24cm) cutout width. Overall width 21.5" (55cm), pad extends back 3.75" (10cm) posterior from current table edge for additional upper body/extremity support.



PB3005 Form to Fit Pad-8"Hx11"W PB3007 Form to Fit Pad-8"Hx13"W

Form to fit pad with built in lateral support that can be custom fitted to the user. Pad size for PB3005 is 8"Hx11"W (20x28cm) and PB3007 is 8"Hx13"W (20x33cm).



PB3031, PB3032 Hygienic Covers

Washable slip on cover available in six different patterns. For form to fit pads PB3005 and PB3007.



#### PT50160 Elbow Pad-11" Cutout

Additional padded support surface on the black molded or clear tray. 11" (28cm) cutout width. Overall width 21.5" (55cm), pad extends back 3.75" (10cm) posterior from current table edge for additional upper body/extremity support.



#### PB5544, PB5546 Lateral Supports

Keeps the upper torso in symmetrical alignment. Adjustable from PB5544 7-11" (18-28cm) and PB5546 10-14" (25-36cm) with a 3.5" (9cm) height range. Pad size is 4"Hx6"W (10x15cm). For planar pads only.



PB3049 Hygienic Cover for PB5588/PB5590

Washable slip on cover available in six different patterns. For PB5588 and PB5599.



#### PY5582 Elbow Pad-13" Cutout

Additional padded support surface on the black molded or clear tray. 13" (33cm) cutout width. Overall width 21.5" (55cm), pad extends back 3.75" (10cm) posterior from current table edge for additional upper body/extremity support.



PB5548 Lateral Supports with Elbow Stop

Works as a lateral support and elbow rest. Great for the supine position. Range: 10"-17" with planar pad B3001 and 12"-19" with planar pad B3003. For planar pads only.



PY5626 Head Support 5"Hx8"W PY5628 Head Support 6"Hx10"W PY5630 Head Support 7"Hx14"W

Cushion sizes are: PY5626 5"Hx8"W (13x20cm), PY5628 6"Hx10"W (15x25cm), PY5630 7"Hx114"W (18x36cm). Depth range is +1" to -2.75" (+2 to -7cm) from backrest. Maximum height from top of upper body pad is 10" (25cm).



#### PNG30295 Padded Tray Cover

Washable slip on cover is padded and available in six different patterns.



PB5550

Lateral Supports w/Elbow Stop/Arm Rest
Works as a lateral support, elbow rest and armrest. Great for the supine, upright and prone position. For planar pads only.



#### PA5586 Head Support-Form to Fit

Cushion size 5"Hx10"W (13x25cm). Depth range is -2.5" (6cm) to +1.5" (4cm) from backrest. Maximum height from top of upper body pad is 10" (25cm).



#### PNG50042 Hand Grips

Provides additional arm and hand support and alignment. Easily movable for precise arm positioning. Attaches with suction cups. Clear tray required.



PB5552 Elbow Stop with Arm Rest

Great for loading and bringing to upright. Works with both planar and form to fit upper body supports. For planar and form to fit pads.



PA3035, PA3036, PB3033, PA3037 Hygienic Cover

Washable slip on cover available in six different patterns. For PY5626, PY5628 PY5630 and PA5586.



#### PA5594 Push Handle

Makes pushing the stander easier for the caregiver and also aids in lifting and lowering the child. Required for dual control.



PNG30170 X-Style Chest Vest-9.5"Lx9"W PNG30056 X-Style Chest Vest-11"Lx9.5"W

Neoprene® with camlock adjustment provides anterior trunk support for child.



#### PB5554 Black Molded Tray-21"Lx24"W

Tray is accessible in all positions and provides anterior support as user moves from supine to upright and prone positions. Adjustable in depth and height. Tray size is 21"Lx24"W (53x61cm).



#### P80809 Tool Pouch

Holds the assembly tools provided with the stander. Attaches to unit with Velcro® straps.



PNG30171 Y-Style Chest Vest-17.5"Lx11.5"W PNG30021 Y-Style Chest Vest-20.5"Lx12.5"W

Neoprene® with camlock adjustment provides anterior trunk support for child.



#### PB5556 Black Molded Swing-Away Tray-21"Lx24"W

Swing-away tray is accessible in all positions and provides anterior support as user moves from supine to upright and prone positions. Adjustable in depth, height and tray angle. Tray size is 21"Lx24"W (53x61cm).



#### P80246 Angle Locator

Helps therapists monitor standing progress. Attaches to the mast. Can be used on leg abduction.

## Zing Supine/MPS Tilt Table (TT) Size 2-Options

# Size 2 154 lbs (70 kg)

#### The Supine/MPS (multi-position stander) TT base includes:

- Four Swivel Locking Casters
- Foot Operated Gas Spring or Pow'r Up Lift
- Mast Receiver with Prone Position Lockout (MPS Only)



#### PB5564 Zing TT Supine Base Gas Spring Lift PB5562 Zing TT MPS Base Gas Spring Lift

Comes with four swivel locking casters, foot operated gas spring, mast receiver. MPS base comes with prone position lockout.



#### PB5518 Platform Foot Plate

Adjustable in height. Foot holders and foot straps can be added for more positioning of the feet.



#### PB5528 Calf/Knee Pad with Strap

17.5"x5.5" (44x14cm) pad attached to a 2" (5cm) belt with a D-Ring Velcro® closure, provides calf/knee stability for user. Fits circumference up to 28"-38" (71-97cm). Gives support from the back of the calf/knee. Adiustable in height and fore/aft.



## PB5568 Zing Supine Base Pow'r Up Lift PB5566 Zing MPS Base Pow'r Up Lift

Comes with four swivel locking casters, Pow'r Up Lift, mast receiver. MPS base comes with prone position lockout.



#### PB5520, PB5522, PB5524 Multi-Adjustable Foot Plates

Foot plates are adjustable forward/aft +/-.75" (2cm) +extra 1.5" (3.8cm), plantar/dorsi +/- 20°, toe-in/toe-out, swivel 30° left/right, and in height. Foot holder sizes: PB5520 7.75"Lx3.25"W (20x8cm), PB5522 9.75"Lx4"W (25x10cm). PB5524 11.75"Lx5"W (30x13cm).



#### PB5576, PB5578, PB5580 Knee Strap

4"x6" (10x15cm) pad attached to a 1.5" (4cm) belt. Fits circumference up to 15" (38cm). Adjustable in height. For leg supports PB5570, PB5572, PB5574.



#### PB5512 Mast

Mast is the base for positioning options including foot plates, knee supports, pelvic supports, upper body supports and head supports.



#### PA5542, PA5544, PB5526 Foot Holders for Platform Foot Plate

Use with the platform foot plate, gives added positioning of users feet. Foot holder sizes: PA5542 7.75"Lx3.25"W (20x8cm),PA5544 9.75"Lx4"W (25x10cm), PB5526 11.75"Lx5"W (30x13cm).



#### PB5530, PB5532, PB5534 Multi-Adjustable Knee Pads

Multi-adjustable knee pads are adjustable in swivel, height +/-.75" (2cm), PA5530 inside width 3.25" (8cm), PA5532 inside width 4.25" (11cm) and PB5534 inside width 5" (13cm). With a depth range of 7" (18cm).



#### PB5514 Mast with Leg Abduction

Mast is the base for support and positioning components and allows for 0-30° of independent leg abduction. Multi-adjustable knee pads and multi-adjustable foot plates required to do leg abduction.



#### PT50082 Secure Foot Straps-10"L PY5638 Secure Foot Straps-15"L PNG50506 Secure Foot Straps-21"L

Velcro® and D-ring adjustment holds feet securely in place at ankles and toes with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



#### PA5678 Multi-Adjustable Calf Pads 4.5" PB5536 Multi-Adjustable Calf Pads 5.5"

Great for adapting to knee contractures. Gives support from the back of the calf. Adjustable up and down with knee pad and fore/aft 4" (10cm) independently. Must order multi-adjustable knee pads.



#### PB5516 Dual Control

Allows both foot and hand control of the gas spring lift. Must order push handle PB5560.



#### PT50080 Foot Straps-10"L PY5636 Foot Straps-15"L PNG50505 Foot Straps-21"L

Velcro® and D-ring adjustment holds feet in alignment with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlao.



#### PB3012, PB3015, PB3018 Planar Pad

Cushioned pad covered with Neoprene®. Pad sizes: PB3012-19"Hx16"W (48x41cm)

PB3015-22"Hx16"W (56x41cm) PB3018-25"Hx16"W (64x51cm)



#### PA5614 Gas Spring Lift Lockout

Locks gas spring in any position from lying down to fully upright, supine and prone positions.



#### PB5570, PB5572, PB5574 Leg Support Planar Pads

Cushioned Neoprene® pad for leg support. Pad sizes: PB5570-7.5"Hx9"W (19x23cm) PB5572-7.5"Hx12"W (19x30cm) PB5574-7.5"Hx15"W (19x38cm)



#### PB3035, PB3036, PB3037 Hygienic Cover

Washable slip on cover available in six different patterns. For planar pads PB3012, PB3015, PB3018.



#### PB5510 Directional Locking Caster

Adjustable in height. Foot holders and foot straps can be added for more positioning of the feet.



#### PB5600, PB5602, PB5604 Hygienic Covers

Washable slip on cover available in six different patterns. For planar pads PB5570, PB5572, PB5574.



#### PB5584, PB5586 Hip Supports

Keeps the hips in symmetrical alignment. Adjustable from PB5538 7"-13" (18-33cm) and PB5540 10"-1" (25-41cm) with fore/aft 4" (10cm). Pad size is 4.5"x4" (11x10cm).



PT30066, PA3062 Pelvic Positioning Strap

Keeps the upper torso in symmetrical alignment. Fits circumference up to PT30066 35" (89cm) and PA3062 45" (114cm) with a 3.5" (9cm) height range. Pad size is 3.5"Hx3.5"W (9x9cm).



#### PA5586 Head Support-Form to Fit

Cushion size 5"Hx10"W (13x25cm), Depth range is -2.5" (6cm) to +1.5" (4cm) from backrest. Maximum height from top of upper body pad is 10" (25cm).



#### PY5582 Elbow Pad-13" Cutout

Additional padded support surface on the black molded or clear trav. 13" (33cm) cutout width. Overall width 21.5" (55cm), pad extends back 3.75" (10cm) posterior from current table edge for additional upper body/extremity support.



#### PB5592, PB5594 Pelvic Padded Positioning Strap

6"x10" (15x25cm) pad attached to a 2" (5cm) belt. PB5592 fits circumference up to 35" (89cm), PB5594 fits circumference up to 45" (114cm).



#### PB3038 Hygienic Cover for Head Support Planar Pad

Washable slip on cover available in six different patterns. For PB5582.



#### PNG30295 Padded Tray Cover

Washable slip on cover is padded and available in six different patterns.



#### PB5588, PB5590 Lateral Supports

Keeps the upper torso in symmetrical alignment. Adjustable from PB5588 7-11" (18-28cm) and PB5590 10-16" (25-41cm) with a 3.5" (9cm) height range. Pad size is 4"Hx6"W (10x15cm). For planar pads only.



#### PA3035, PA3036, PB3033 Hygienic Covers

Washable slip on cover available in six different patterns. For PY5626, PY5628 and PY5630.



#### PNG50042 Hand Grips

Provides additional arm and hand support and alignment. Easily movable for precise arm positioning. Attaches with suction cups. Clear tray required.



#### PNG30170 X-Style Chest Vest-11"Lx9.5"W PNG30056 X-Style Chest Vest-14.5"Lx9.5"W

Neoprene® with camlock adjustment provides anterior trunk support for child.



#### PA3037 Hygienic Cover

Washable slip on cover available in six different patterns. For PA5586.



#### PA5594 Push Handle

Makes pushing the stander easier for the caregiver and also aids in lifting and lowering the child. Required for dual control.



PNG30171 Y-Style Chest Vest-17.5"Lx11.5"W PNG30021 Y-Style Chest Vest-20.5"Lx12.5"W

Neoprene® with camlock adjustment provides anterior trunk support for child.



#### PB5554 Black Molded Tray-21"Lx24"W

Tray is accessible in all positions and provides anterior support as user moves from supine to upright and prone positions. Adjustable in depth and height. Tray size is 21"Lx24"W (53x61cm).



#### P80809 Tool Pouch

Holds the assembly tools provided with the stander. Attaches to unit with Velcro® straps.



PB5596, PB5598 Upper Body Padded Positioning Strap

6"x10" (15x25cm) pad attached to a 2" (5cm) belt. PB5596 fits circumference up to 35" (89cm), PB5598 fits circumference up to 45" (114cm).



#### PB5556 Black Molded Swing-Away Tray-21"Lx24"W

Swing-away tray is accessible in all positions and provides anterior support as user moves from supine to upright and prone positions. Adjustable in depth, height and tray angle. Tray size is 21"Lx24"W (53x61cm).



#### P80246 Angle Locator

Helps therapists monitor standing progress. Attaches to the mast. Can be used on leg abduction.




#### PB3049 Hygienic Cover

Washable slip on cover available in six different patterns. For PB5588 and PB5599.



#### PB5558 Clear Swing Away Tray-21"Lx24"W

Swing-away tray is accessible in all positions and provides anterior support as user moves from supine to upright and prone positions. Adjustable in depth, height and tray angle. Tray size is 21"Lx24"W (53x61cm).

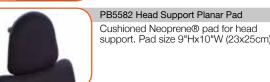

support. Pad size 9"Hx10"W (23x25cm).



#### PT50156 Elbow Pad-9.25" Cutout

Additional padded support surface on the black molded or clear tray. 9.25" (24cm) cutout width. Overall width 21.5" (55cm), pad extends back 3.75" (10cm) posterior from current table edge for additional upper body/extremity support.





#### PT50160 Elbow Pad-11" Cutout

Additional padded support surface on the black molded or clear tray. 11" (28cm) cutout width. Overall width 21.5" (55cm), pad extends back 3.75" (10cm) posterior from current table edge for additional upper body/extremity support.

## Size 2 ...09-...0



#### The Prone base includes:

- Four Swivel Locking Casters
- Foot Operated Gas Spring or Pow'r Up Lift
- Mast Receiver



#### PB5606 Zing Prone Base Gas Spring Lift

Comes with four swivel locking casters, foot operated gas spring, and a mast receiver.



#### PB5612 Platform Foot Plate

Adjustable in height. Foot holders and foot straps can be added for more positioning of the feet.



#### PB5636 Knee Strap

4"x6" (10x15cm) pad attached to a 1.5" (4cm) belt. Fits circumference up to 15" (38cm). Adjustable in height. For PB5622, PB5624, and PB5626.



#### PB5608 Zing Prone Base Pow'r Up Lift

Comes with four swivel locking casters, Pow'r Up Lift, mast receiver.



#### PB5616, PB5618, PB5620 Multi-Adjustable Foot Plates

Foot plates are adjustable forward/aft +/-.75" (2cm) +extra 1.5" (3.8cm), plantar/dorsi +/- 20°, toe-in/toe-out, swivel 30° left/right, and in height. Foot holder sizes: PB5616 7.75"Lx3.25"W (20x8cm), PB5618 9.75"Lx4"W (25x10cm), PB5620 11.75"Lx5"W (30x13cm).



#### PB3000 Planar Pad-10"Hx11"W PB3002 Planar Pad-10"Hx13"W

Cushioned pad covered with Neoprene®. Pad sizes: PB3000 10"Hx11" (25x28cm) PB3002 10"Hx13"W (25x33cm).



#### PB5512 Mast

Mast is the base for support and positioning components.



#### PA5542, PA5544, PB5526 Foot Holders for Platform Foot Plate

Use with the platform foot plate, gives added positioning of users feet. Foot holder sizes: PA5542 7.75"Lx3.25"W (20x8cm),PA5544 9.75"Lx4"W (25x10cm), PB5526 11.75"Lx5"W (30x13cm).



## PB3004 Form to Fit Pads-10"Hx11"W PB3006 Form to Fit Pads-10"Hx13"W

Form to fit pad can be custom fitted to the user. Pad sizes: PB3004 10"Hx11" (25x28cm ) PB3006 10"Hx13"W (25x33cm).



#### PB5514 Mast with Leg Abduction

Mast is the base for support and positioning components and allows for 0-30° of independent leg abduction. Multi-adjustable knee pads and multi-adjustable foot plates required to do leg abduction.



#### PT50082 Secure Foot Straps-10"L PY5638 Secure Foot Straps-15"L PNG50506 Secure Foot Straps-21"L

Velcro® and D-ring adjustment holds feet securely in place at ankles and toes with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



#### PB5538, PB5540 Hip Supports

Keeps the hips in symmetrical alignment. Adjustable from PB5538 7"-13" (18-33cm) and PB5540 12"-18" (30-46cm) with fore/aft 4" (10cm). Pad size is 4.5"x4" (11x10cm). For planar pads only.



#### PB5640 Dual Control

Allows both foot and hand control of the gas spring lift. Must order push handle PB5632.



#### PT50080 Foot Straps-10"L PY5636 Foot Straps-15"L PNG50505 Foot Straps-21"L

Velcro® and D-ring adjustment holds feet in alignment with multiple attachment slots for proper foot positioning. Straps measured in length over top of foot, with approximately 1.5" (4cm) of Velcro® overlap.



T II CHIE

#### PA3011, PT30066 Pelvic Positioning Strap

2" (5cm) wide belt has a D-Ring Velcro® closure, provides hip stability for user. Fits circumference up to PA3011 29" (74cm) and PT30066 35" (89cm).



#### PA5614 Gas Spring Lift Lockout

Locks gas spring in any position from lying down to fully upright and supine positions.



#### PB5528 Calf/Knee Pad with Strap

17.5"x5.5" (44x14cm) pad attached to a 2" (5cm) belt with a D-Ring Velcro® closure, provides calf/knee stability for user. Fits circumference up to 28"-38" (71-97cm). Gives support from the back of the calf/knee. Adjustable in height and fore/aft.



#### PA5610, PB5542 Pelvic Padded Positioning Strap

6"x10" (15x25cm) pad attached to a 2" (5cm) belt. Fits circumference up to PA5610 29" (74cm) and PB5542 35" (89cm).



#### PB5510 Directional Locking Caster

Great for transporting stander. Directionally locks one caster while leaving the other three free to swivel.



#### PB5622, PB5624, PB5626 Multi-Adjustable Knee Pads

Multi-adjustable knee pads are adjustable in swivel, height +/-.75" (2cm), PA5622 inside width 3.25" (8cm), PA5624 inside width 3.25" (11cm) and PB5626 inside width 5" (13cm). With a depth range of 7" (18cm).



#### PB3025, PB3026 Hygienic Cover

Washable slip on cover available in six different patterns. For planar pads PB3000 and PB3002.



PB3027, PB3028 Hygienic Cover

Washable slip on cover available in six different patterns. For form to fit pads PB3004 and PB3006.



PA5710 Hygienic Cover

Washable slip on cover available in six different patterns. For face apertures PB5638.



#### P80809 Tool Pouch

Holds the assembly tools provided with the stander. Attaches to unit with Velcro® straps.



#### PA3006, PA3007, PA3041, PA3042 Planar Pad

Cushioned pad covered with Neoprene®. Pad

SIZES: PA3006-5"Hx7"W (13x18cm) PA3007-5"Hx9"W (13x21cm) PA3041-7"Hx7"W (18x18cm) PA3042-7"Hx9"W (18x21cm)



#### PB5628 Black Molded Tray-21"Lx24"W

Tray provides anterior support. Adjustable in depth and height. Tray size is 21"Lx24"W (53x61cm).



#### P80246 Angle Locator

Helps therapists monitor standing progress. Attaches to the mast. Can be used on leg abduction.



#### PA3079, PB3046, PA3081, PB3047 Form to Fit Pads

Form to fit pad with extra sternum positioning. Built in lateral support that can be custom fitted to the user. Pad sizes are:

PA3079-5"Hx12"W (13x30cm) PB3046-6"Hx13"W (15x33cm PA3081-7"Hx12"W (18x30cm) PB3047-10"Hx13"W (25x33cm



#### PB5630 Clear Acrylic Tray-21"Lx24"W

Tray provides anterior support. Adjustable in depth, height and tray angle. Tray size is 21"Lx24"W (53x61cm).



#### PB5627 Lateral Supports

Keeps the upper torso in symmetrical alignment. Adjustable range of 7"-12" (18-30cm) with 7"W planar pads, 9"-14" (23-36cm) with 9"W planar pads. For planar pads only.



#### PT50156 Elbow Pad-9.25" Cutout

Additional padded support surface on the black molded or clear tray. 9.25" (24cm) cutout width. Overall width 21.5" (55cm), pad extends back 3.75" (10cm) posterior from current table edge for additional upper body/extremity support.



#### PA5610, PB5542 Upper Body Padded Positioning Strap

6"x10" (15x25cm) pad attached to a 2" (5cm) belt. PA5610 fits circumference up to 29" (74cm), PB5542 fits circumference up to 35" (89cm).



#### PT50160 Elbow Pad-11" Cutout

Additional padded support surface on the black molded or clear tray. 11" (28cm) cutout width. Overall width 21.5" (55cm). pad extends back 3.75" (10cm) posterior from current table edge for additional upper body/extremity support.



#### PA3028, PA3029. PA3045, PA3046 Hygienic Cover

Washable slip on cover available in six different patterns. For planar pads PA3006, PA3007, PA3081, and PA3042.



#### PY5582 Elbow Pad-13" Cutout

Additional padded support surface on the black molded or clear tray. 13" (33cm) cutout width. Overall width 21.5" (55cm). pad extends back 3.75" (10cm) posterior from current table edge for additional upper body/extremity support.



#### PA3086, PB3050, PA3088, PB3051 Hygienic Cover

Washable slip on cover available in six different patterns. For form to fit pads PA3079, PB3046, PA3081, and PB3047.



#### PT30093 Padded Tray Cover

Washable slip on cover is padded and available in six different patterns.



#### PB3049 Hygienic Covers for Lateral Supports

Washable slip on cover available in six different patterns. For lateral supports PA5627.



#### PNG50042 Hand Grips

Provides additional arm and hand support and alignment. Easily movable for precise arm positioning. Attaches with suction cups.



#### PB5638 Face Aperture-9"H

Pad size is 9"Hx3"W (23x8cm). With a 7" (18cm) range from the upper body support, individual face aperture pads are adjustable. Supports the head.



#### PB5632 Push Handle for Tray

Makes pushing the stander easier for the caregiver and also aids in lifting and lowering the child. Required for dual control.



## EasyStandij

Altimate Medical, Inc.

262 West First Street Morton, MN 56270 USA

Tel: 800.342.8968 • 507.697.6393

Fax: 877.342.8968 • 507.697.6900

info@easystand.com • easystand.com